


# BRUNO

## Wheelchair-Lifters

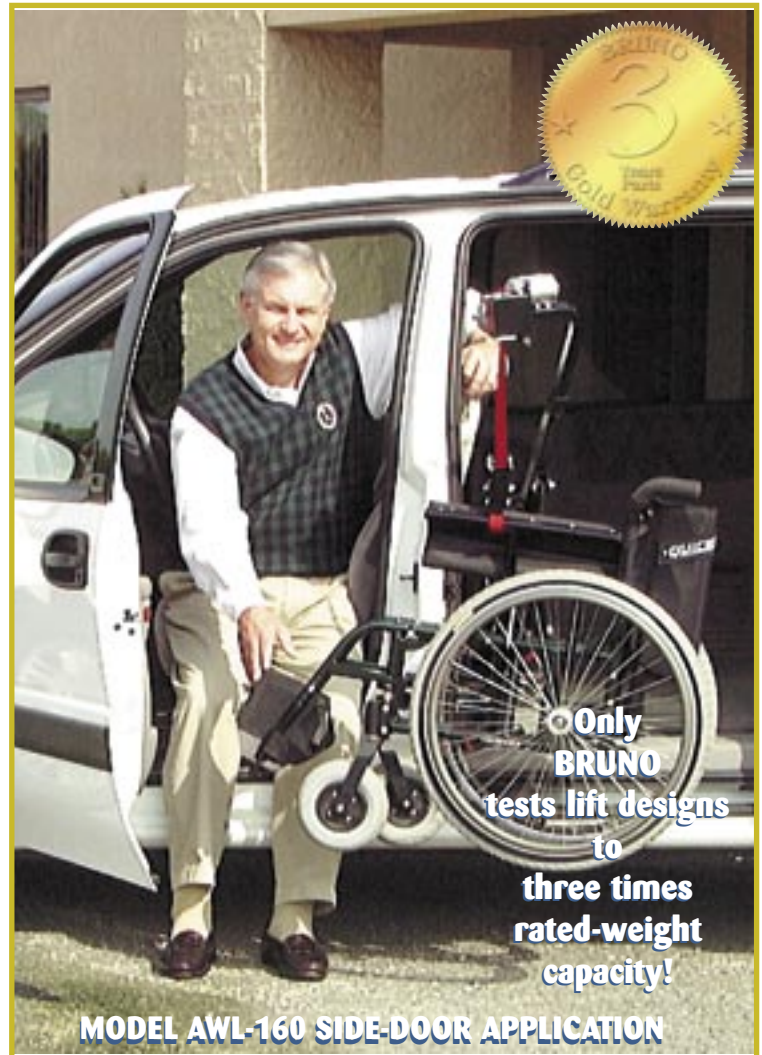

- ◆ The Bruno Wheelchair-Lifter Vehicle Lifts, Models AWL-150 and AWL-160, eliminate the struggle of lifting/storing your folding wheelchair into a full-size van or minivan.
- ◆ The Wheelchair-Lifter Models AWL-150 and AWL-160 raise a wheelchair vertically into the rear or side door of your full-size van or minivan in just seconds – all at the touch of a button. Plus, Bruno's additional Wheelchair-Lifter Model AWL-100 can lift and store your wheelchair into the trunk of a car.
- ◆ These electrically-operated lifts are powered by the vehicle's battery; they can also be used with Bruno's optional Pow'rPac. With the wheelchair next to the bumper or side door, just attach the chair to the lift's docking device and push the **UP** button to raise the wheelchair above the bottom of the door opening. Then, simply swing the lift arm and wheelchair into the vehicle and push the **DOWN** button—it's as simple as that!
- ◆ The Wheelchair-Lifter Models AWL-150 and AWL-160 can be installed in most full-size vans and minivans as long as the storage area is large enough to accommodate your folding wheelchair. Please see the back of this brochure for a step-by-step illustration of how the Wheelchair-Lifter Models AWL-150 and AWL-160 work.

### OPTIONAL BRUNO POW'R PAC



- ◆ Easy to use!
- ◆ Convenient!
- ◆ Works in most minivans, SUVs, station wagons & sedans!

Pow'rPac easily installs in your vehicle – just plug into the vehicle's power outlet!

The complete, independent power supply for your automotive lift!

- ◆ **EXCLUSIVE!** Bruno's R<sup>3</sup> Lift Belt is 50% stronger! Count on Bruno's R<sup>3</sup> Lift Belt for ease of use, longer life and safer operation – **another Industry First!**
- ◆ Expert installation and complete, after-sale service is provided by Bruno's nationwide network of dealers.
- ◆ Only Bruno® has over 25 different interior/exterior scooter, wheelchair and power chair lifts for nearly any sedan, hatchback, station wagon, convertible, minivan, Sport Utility Vehicle (SUV), full-size van, pickup truck and the new crossover vehicles.
- ◆ Plus, all Bruno lifts are eligible for reimbursement under the terms of Mobility Programs offered by GM, Saturn, Ford, DaimlerChrysler/Dodge, Toyota, Lexus, Volvo and Volkswagen.



# Using a *Bruno Wheelchair-Lifter*<sup>®</sup> is easy!



Once you are situated inside or at the back of the vehicle, fold the wheelchair up. Next, unlock and swing the lift arm out of the vehicle. Note: Make sure the side door or rear hatch are fully open before swinging the lift out.



Lower the lift belt and attach the lifting device to the wheelchair.



Push the **UP** button until all of the wheelchair is above the bottom of the door opening.



Swing the wheelchair into the van and push the **DOWN** button to lower the wheelchair into the van.



The lift and wheelchair fit snugly in your van. In some instances, bucket or rear-seats may remain in the van.



## **WHEELCHAIR-LIFTER Trunk Application Model AWL-100**



The Lift weight rating is the MAXIMUM load value. Factors inherent to specific applications including, but not limited to, vehicle structure and condition, operating environment, and customer ability may reduce the effective weight rating.



**BRUNO**<sup>®</sup>  
INDEPENDENT LIVING AIDS



**Bruno Independent Living Aids, Inc.**

1780 Executive Drive

P.O. Box 84, Oconomowoc, WI 53066

(262)567-4990 ■ FAX (262) 953-5501

<http://www.bruno.com>

**Bruno lifts are eligible for reimbursement under the terms of Mobility Programs offered by GM, Saturn, DaimlerChrysler/Dodge, Ford, Toyota, Lexus, Volvo and Volkswagen.**

**Represented By:**

